



Diagnostic Monitor

RadiForce®RX250



A 2 megapixel high-brightness color monitor ideal for the accurate display of CT, MRI, and CR grayscale images and color images such as 3D color rendering and nuclear medicine scans. Featuring a new design and unique EIZO technology, this monitor provides both comfort and image precision to medical professionals.

- Clearly defined images using EIZO's Sharpness Recovery technology
- New user-friendly design with fresh, clean aesthetic
- Reduced size and thin bezels that save workspace
- Reduced power consumption with energy-efficient LED backlight
- · Reduced reflections for image clarity in environments with bright ambient lighting
- Hassle-free multi-monitor configuration using a daisy chain
- · Effortless quality control with built-in calibration sensor
- Stable images across the screen with brightness uniformity
- Power conservation with integrated presence sensor





RadiForce®RX250

Sharpness Recovery
All high-brightness LCD panels exhibit a decrease in sharpness of the original image due to aperture ratio of the pixels becoming larger.
EIZO's unique technology called Sharpness Recovery restores lost information in contours, resulting in an image shown with maximum clarity.

Create the Ideal Environment

The black front bezels are ideal for viewing the screen in dark reading rooms, making it easier to focus on images, while the original white stripe around the sides of the monitor presents a fresh, clean aesthetic.

Save Work Space with Sleek Cabinet Design

The monitor's width, height, and depth were reduce by 15 mm, 37.5 mm, and 45.5 mm respectively – 27% less space than its predecessor, saving more workspace for other tasks. Reducing the width of the monitor's frame reduces eye movement when comparing side-by-side images in a dual monitor configuration.

Save on Power Consumption

The monitor uses a new panel equipped with an energy-efficient LED backlight which reduces power consumption by 27% compared to its predecessor.

EIZO's internal measurement under the recommended brightness of 400 cd/m².

Reduce Reflections for Image Clarity

Anti-reflection (AR) coating greatly reduces reflections caused by outside light without affecting the light emitting from the monitor. It also minimizes the loss of contrast that causes blacks to appear washed out with an anti-glare (AG) coating. This makes AR coating ideal for environments with bright ambient lighting.

Hassle-Free Multi-Monitor Configuration

Using the DisplayPort connection, you can drive several monitors in a daisy chain sequence. This allows you to configure a multi-monitor setup without the complicated hassle of excessive cabling.

Manage Effortless Quality Control

An Integrated Front Sensor (IFS) housed within the front bezel measures brightness and grayscale tones and calibrates to the DICOM Part 14 standard. The hands-free IFS performs quality control tasks and does not interfere with the viewing area while in use. This dramatically cuts the workload and maintenance costs needed for maintaining monitor quality control.

Attain Steady Images Across the Screen

The Digital Uniformity Equalizer (DUE) function helps to even out fluctuations in brightness and chroma on different parts of the screen to provide smoother images, a quality typically difficult to attain due to the characteristics of LCD monitors.


Conserve Energy While Away

A presence sensor prompts the monitor to switch to power save mode when the user is away and resumes operation upon their return to conserve power when not in use. In the latest RadiForce models, the sensor detects the user's presence using a heat sensing method. This increases the range and angle of detection compared to conventional detection methods, making it effective when using multiple monitors.

Specifications

Model Variations Cabinet Color		RX250: Anti-Glare coating RX250-AR: Anti-Reflection coating Black
Backlight	LED	
Size	54 cm / 21.3"	
Native Resolution	1200 x 1600 (3:4 aspect ratio)	
	Viewable Image Size (H x V)	324.0 x 432.0 mm
	Pixel Pitch	0.270 x 0.270 mm
	Display Colors	10-bit colors (DisplayPort): 1.07 billion
	Display Colors	(maximum) colors
		8-bit colors: 16.77 million from a palett
		of 68 billion colors
	Viewing Angles (H / V, typical)	178° / 178°
	Brightness (typical)	800 cd/m ²
	Recommended Brightness for Calibration	400 cd/m ²
	Contrast Ratio (typical)	1400:1
	Response Time (typical)	20 ms (on / off)
Video	Input Terminals	DVI-D x 1, DisplayPort x 1
USB Power	Output Terminals	DisplayPort x 1 (daisy chain)
	Digital Scanning Frequency (H / V)	31 - 100 kHz / 59 - 61 Hz
	Digital Scalling Frequency (H / V)	Frame synchronous mode: 59 - 61 Hz
	Function	
	Standard	1 upstream, 2 downstream USB 2.0
		AC 100 - 240 V: 50 / 60 Hz
	Power Requirements	79 W
	Maximum Power Consumption	· · · · · · · · · · · · · · · · · · ·
	Typical Power Consumption	38 W
	Power Save Mode	Less than 1 W
Sensor	Power Management	DVI DMPM, DisplayPort 1.2a
Sensor		Backlight Sensor, Integrated Front
		Sensor, Presence Sensor, Ambient
	D: 11 0: 12: 12	Light Sensor
	Brightness Stabilization	Yes
& Functions	Digital Uniformity Equalizer	Yes
		CAL Switch
OSD Languages		English, German, French, Italian,
		Japanese, Simplified Chinese, Spanis
		Swedish, Traditional Chinese
	Net Weight	8.2 kg
	Net Weight (Without Stand)	5.4 kg
cations Hole Spacing (VESA Standard) Certifications & Standards		100 x 100 mm
(Please contact the EIZO group		CE (Medical Device Directive),
company or distributor in your country for the latest information.)		EN60601-1, ANSI/AAMI ES60601-1,
		CSA C22.2 No. 601-1, IEC60601-1,
	,	VCCI-B, FCC-B, CAN ICES-3(B), RCI
EDA 540(L) OL		RoHS, China RoHS, WEEE, CCC, EA
FDA 510(k) Clearance		Yes (for General Radiography*)
Supplied Accessories		AC power cord, signal cables (DVI-D
(May vary by country. Please contact EIZO for details.)		DVI-D, DisplayPort - DisplayPort), US
comac	LEIZO IOI GELAIIS.)	cable, Utility Disk (RadiCS LE, PDF
		instructions for use, PDF installation
		manual), instructions for use
Warranty	1	Five Years

^{*}Display of mammography images for diagnosis is not supported.

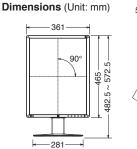





EIZ Corporation

153 Shimokashiwano, Hakusan, Ishikawa 924-8566 Japan Phone +81-76-277-6792 Fax +81-76-277-6793 www.eizoglobal.com

All product names are trademarks or registered trademarks of their respective companies. EIZO, RadiForce, RadiCS, and RadiNET are registered trademarks of EIZO Corporation. Specifications are subject to change without notice.

Copyright@ 2016 EIZO Corporation. All rights reserved. (160202C)



Assured Systems

Assured Systems is a leading technology company with over 1,500 regular clients in 80 countries, deploying over 85,000 systems to a diverse customer base in 12 years of business. We offer high-quality and innovative rugged computing, display, networking and data collection solutions to the embedded, industrial, and digital-out-of-home market sectors.

US

sales@assured-systems.com

Sales: +1 347 719 4508 Support: +1 347 719 4508

1309 Coffeen Ave Ste 1200 Sheridan WY 82801 USA

EMEA

sales@assured-systems.com

Sales: +44 (0)1785 879 050 Support: +44 (0)1785 879 050

Unit A5 Douglas Park Stone Business Park Stone ST15 0YJ United Kingdom

VAT Number: 120 9546 28

Business Registration Number: 07699660